

# Synthetic Biogel® PI OrthoPro®



**Sterile, powder-free, synthetic polyisoprene orthopedic surgical gloves. Thicker for increased protection.**

Biogel® PI OrthoPro® reduces the possibility of glove-related latex protein sensitization because it is made from a synthetic elastomer. The new glove provides fit, feel and comfort comparable to natural rubber latex.

- Made from a biomechanically optimized glove mold designed to reduce hand and thumb fatigue during surgery.
- A specialized manufacturing process enables grip and control of surgical instruments.
- Longer cuff for added protection and improved glove-gown interface.
- Brown in color to potentially reduce glare.

Biogel® PI OrthoPro® is typically 13% thicker than Biogel® PI to provide durability and extra protection during more rigorous and/or longer procedures.

Recommended for all orthopedic surgeries, where robust barrier protection is necessary or surgeries where latex allergies are a concern for patients or clinicians.

## Material information

- Biogel polymer coating
- Micro-roughened surface
- Brown pigment



ACTUAL COLOR

## Re-order # 476

REF	Size	Pairs
47660	6	40/Box
47665	6 ½	40/Box
47670	7	40/Box
47675	7 ½	40/Box
47680	8	40/Box
47685	8 ½	40/Box
47690	9	40/Box

4 boxes per case

## Superior quality with Biogel

- Unique Biogel coating on the inner surface
  - Makes gloves easy to don even with damp hands
  - Soothes the skin to help prevent moisture loss<sup>2</sup>
- Beaded cuff for added security
- Every glove is air inflation tested and visually inspected for quality and safety
- Industry leading AQL freedom from holes of 0.65
- Powder-free to eliminate starch powder related complications

# Biogel Sterile Surgical Gloves; #476

## PRODUCT SPECIFICATIONS

REF	Size	Length, mm (Tolerance + 20 mm; -10 mm)	Lay Flat Palm Width, mm (± 3 mm)
47660	6.0	295	77
47665	6.5	295	85
47670	7.0	298	91
47675	7.5	308	96
47680	8.0	309	103
47685	8.5	311	109
47690	9.0	311	115

4 boxes per case

Typical thickness profile – single wall		
Cuff	9.8 mils	0.255 mm
Palm	12.2 mils	0.315 mm
Finger	13.8 mils	0.350 mm

Physical glove properties	Standard requirement	Biogel
<b>Force at break (N)</b>		
Initial	≥9	Typically 22
Aged	≥9	Typically 17
<b>Typical accelerator analysis % w/w</b>		
Dithiocarbamate (DTC)	n/a	<0.10
Diphenyl thiourea (DPTU)	n/a	<0.03
Diphenyl guanidine (DPG)	n/a	<0.25
Zinc mercaptobenzothiazole (ZMBT)	n/a	<0.05
<b>AQL freedom from holes</b> (1000ml water leak test) Post packing and irradiation Process average typically	1.5	0.65 <0.2%

## General information

**Pyrogenicity:** each batch of Biogel gloves is certified to be either non-pyrogenic or to have a low endotoxin level (<0.5 EU/ml).

**Product standards:** Biogel gloves are tested and manufactured to the following standards:

- **Quality/Environmental:** ISO 9001, ISO 13485, ISO 14001
- **Product:** ASTM D3577, EN455-1, EN455-2, EN455-3, EN455-4
- **Sterilization:** gamma irradiation (20 Kgy [2.0 Mrad]), EN556, ISO 11137
- **Viral penetration:** bacteriophage test, ASTM F1671
- **Allergenicity/Pyrogenicity:** ISO 10993, (PART 5 and 10)/ US Pharmacopoeia USP: (151 & 85)

**Registering authority:** in Europe the gloves are CE marked (notified by body BSI, number 0086) indicating compliance with council directive 93/42/EEC, annex VII and annex V, section 3.2. Biogel surgical gloves are a class IIa product.

**Storage:** store in a cool, dry place away from sources of heat or direct sunlight.

**Packaging:** one pair per pack, in a high quality inner wrap, packed into a film pack (constructed of a laminate of polyester and low-density polyethylene). 40 pairs per collation case for sizes 6.0 – 9.0; 160 pairs per transit case for sizes 6.0 – 9.0.

**Disposal:** gloves & outer wrap dispose of as clinical waste. Paper inner wrap, collation case & transit case can be recycled as paper or disposed of as clinical waste.

**Shelf life:** three (3) years from date of manufacture.

**Manufacturer:** made and packed in Malaysia by Mölnlycke Health Care Sdn Bhd.

**Country of origin:** Malaysia.

**E-mail address:** biogel@molnlycke.com.

**For inquiries:** USA: Norcross GA USA 30092 1-800-843-8497

**Date of issue:** February 2012

References 1. Cochrane Review 2006. 2. Source: Testing conducted at PRAC and overseen by a board-certified dermatologist, 2004. Report 04-007. 3. Using Modified Lowry (EN455) / ASTM D5712.

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

The Mölnlycke Health Care and Biogel names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. ©2012 Mölnlycke Health Care AB. All rights reserved.