



## Donning guide: Surgical mask with tie-band

- A. Pre-form the nose-clip. Place the surgical mask on your face making sure to cover your nose and mouth
- B. Position the upper tie-bands at the crown of your head and tighten securely to minimise air leakage
- C. Position the lower tie-bands at the nape of your neck and tighten
- D. Always unfold the mask fully to minimise the number of layers you have to breathe through
- E. Change surgical mask when contaminated as well as between operations
- F. When there is a risk of fluid splashes use a splash resistant surgical mask and protective eyewear or a splash resistant surgical mask with a visor
- G. Dispose the surgical mask after use

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00. The Mölnlycke and BARRIER trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. ©2020 Mölnlycke Health Care AB. All rights reserved. HQIM001621

  
**Mölnlycke®**