



Donning guide: Surgical mask with visor

If there is a risk of fluid splashes ensure that you select either a four layer splash resistant surgical mask with visor or a four layer splash resistant surgical mask and eyewear.

- A. Before putting on the surgical mask, pre-fold the visor at the edges to achieve optimal protection from side splashes that could potentially come into contact with the eyes
- B. Pre-form the nose-clip
- C. Pull the mask from the top and bottom to fully unfold the pleats of the surgical mask
- D. Place the surgical mask on your face making sure to cover your nose and mouth
- E. Position the upper tie-bands at the crown of your head and tighten securely to minimise air leakage
- F. Position the lower tie-bands at the nape of your neck and tighten
- G. Always unfold the mask fully to minimise the number of layers you have to breathe through
- H. Change surgical mask when contaminated as well as between operations
- I. Dispose the surgical mask after use

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00
The Mölnlycke and BARRIER trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2020 Mölnlycke Health Care AB. All rights reserved. HQIM001622



Mölnlycke®